



# WIDEX EVOKE™ App highlights - Android

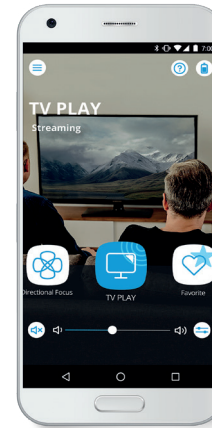


For more information on compatibility go to <https://global.widex.com/en/support/evoked-hearing-aid-app/compatibility>

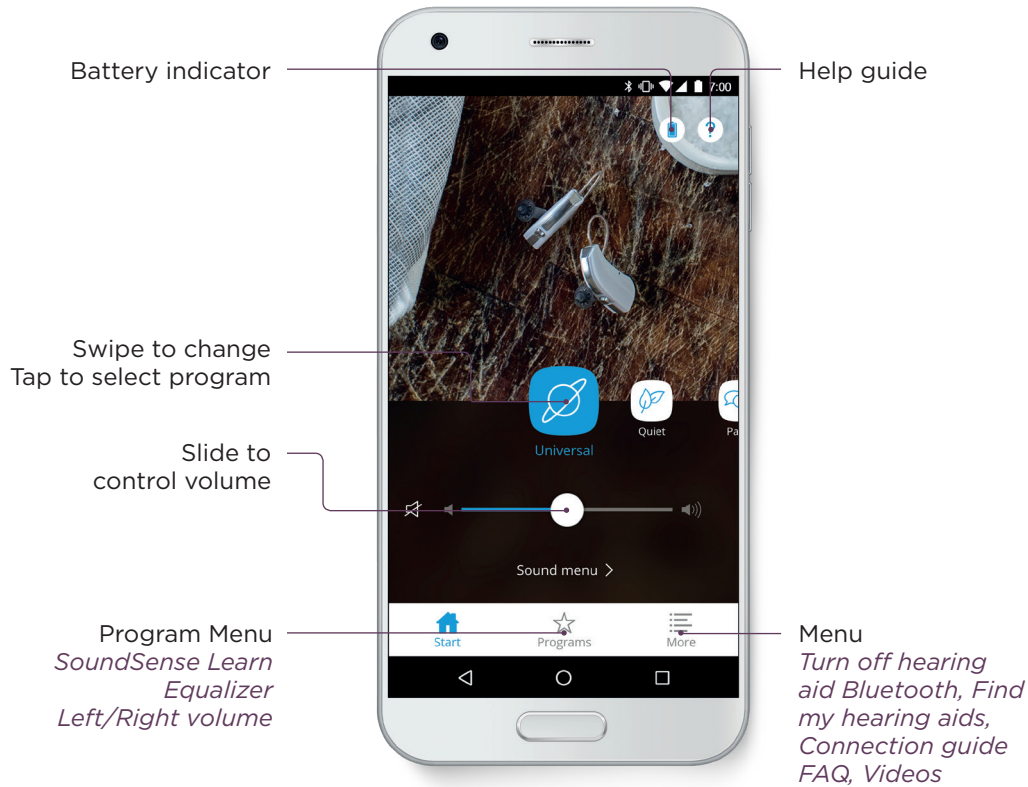


## TV PLAY

Access sound directly from your TV PLAY device and set the volume for your personal comfort.

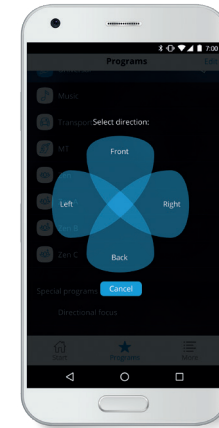


## Sound Direction

Focus your listening attention behind you, or to the left/right with the click of a button.

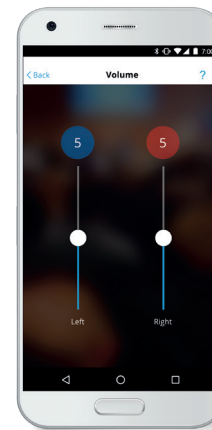

## Volume R/L

Adjust volume on two hearing aids individually.

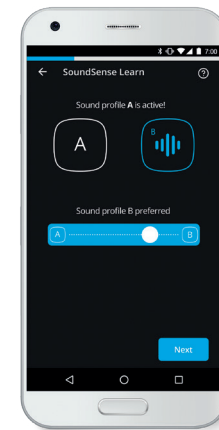


## SoundSense Learn A/B Comparator

Easily customize and save your preferred listening settings in the moment. (Not available E110)


# How to pair FUSION™ 2 hearing aids to an Android phone to use the EVOKE™ App




**STEP 1**  
Go to *Settings*.

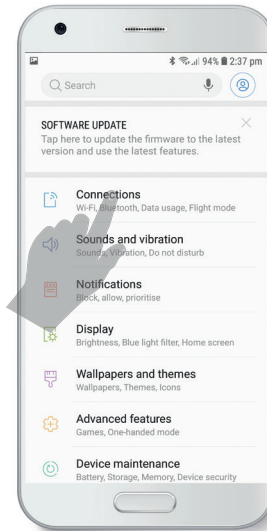


**STEP 2**  
Go to *Connections*.

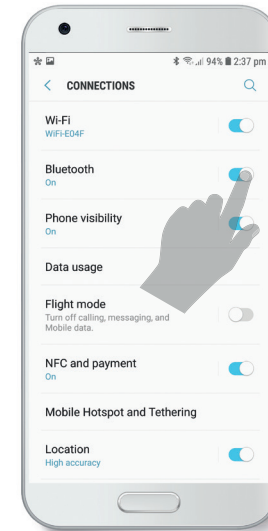


**STEP 3**  
GO to *Bluetooth*.

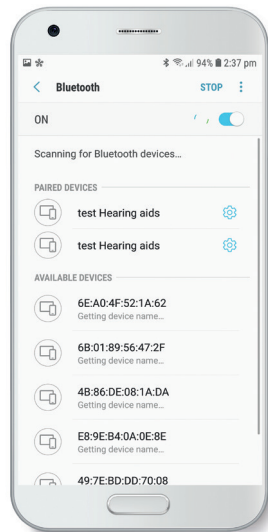

**STEP 4**  
Switch Bluetooth to *on*, then open & close hearing aid battery doors.

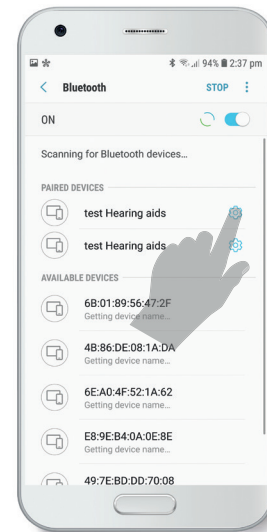

**STEP 5**  
Once both hearing aids show up, pair them one by one, start by clicking on the first one.

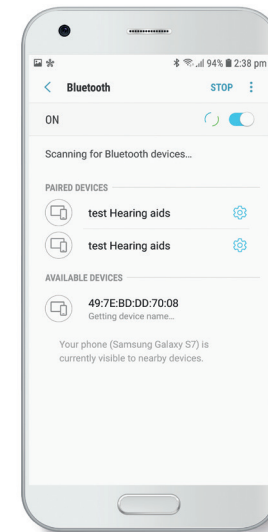

**STEP 6**  
If pairing successful, both devices will show as paired devices. You can now open and use the **EVOKE™** app.



Once the pairing is complete, download the **EVOKE™** App by searching WIDEX **EVOKE™** in the Google Play Store. For more information, go to [www.widex.com](http://www.widex.com)