




# WIDEX EVOKE™ App highlights - iPhone

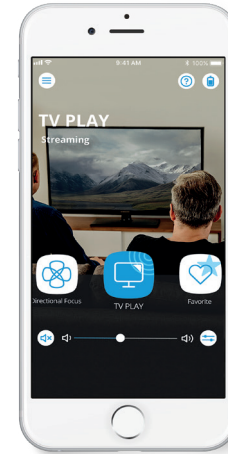


For more information on compatibility go to <https://global.widex.com/en/support/evoked-hearing-aid-app/compatibility>

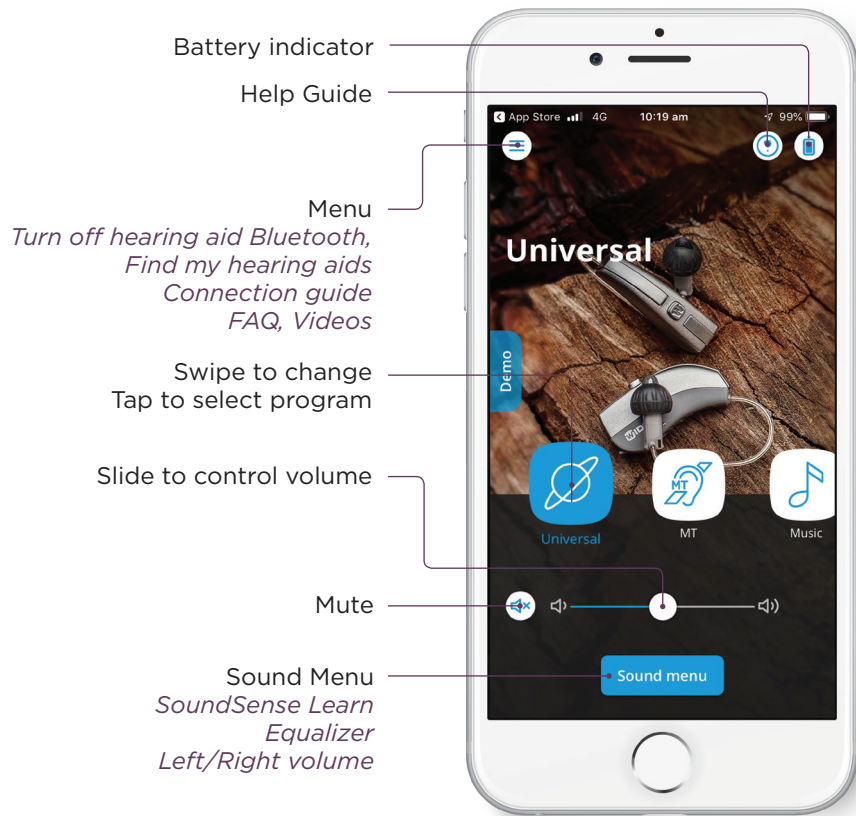


**TV PLAY**  
Access sound directly from your V PLAY device and set the volume for your personal comfort.

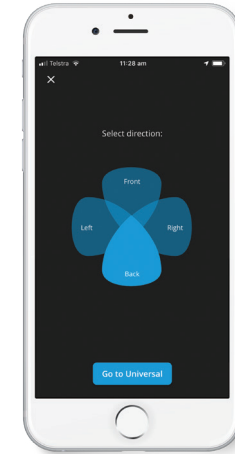

**Sound Direction**  
Focus your listening attention behind you, or to the left/right with the click of a button.

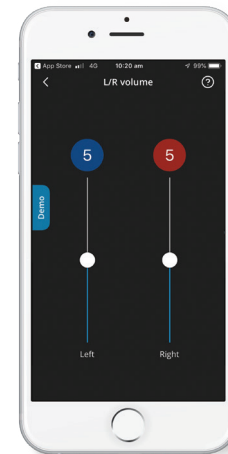


**Volume R/L**  
Adjust volume on two hearing aids individually.

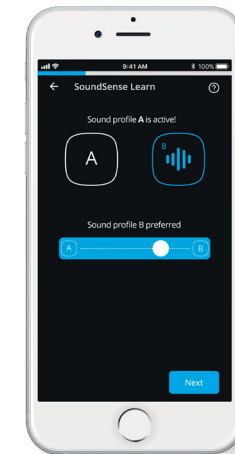


**SoundSense Learn A/B Comparator**  
Easily customize and save your preferred listening settings in the moment. (Not available E110)


# How to connect WIDEX EVOKE™ to an iPhone or iPad

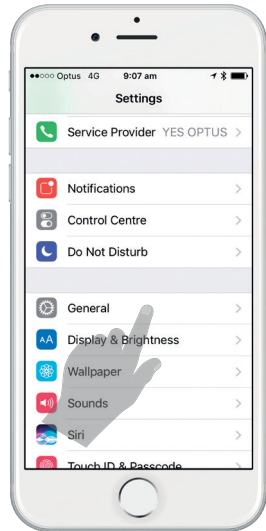


**STEP 1**  
Go to *Settings* and select *General*.

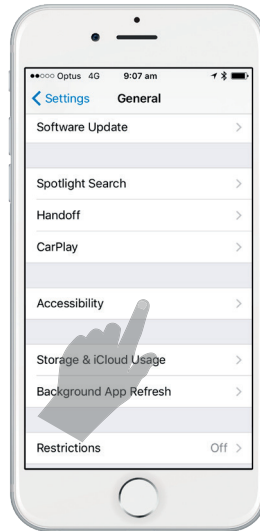

**STEP 2**  
Select *Accessibility*.

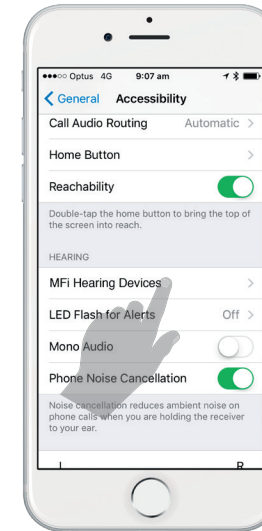


**STEP 3**  
Select *MFi Hearing Devices*

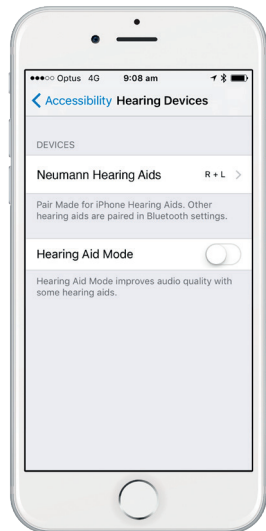


**STEP 4**  
Place the **EVOKE™** devices close to the iPhone, iPad or iPod. Open and close the battery door.

The Apple device will then search for the **EVOKE™** devices. Once connected, select the hearing aids under *Devices*.

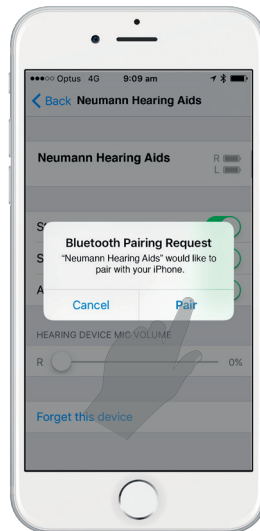

**STEP 5**  
Select *Pair* for each hearing device.

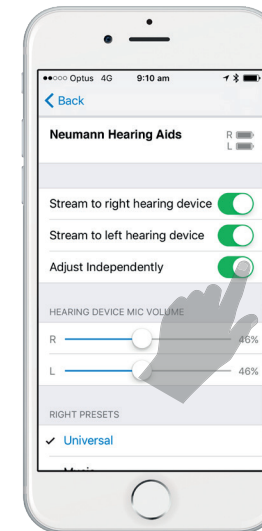

**STEP 6**  
Once complete, the hearing aids will appear.

Congratulations!  
The **EVOKE™** is now paired.

Select *Adjust Independently ON*.



Once the pairing is complete, download the **EVOKE™** App by searching WIDEX **EVOKE™** in the Apple App Store. For more information, go to [www.widex.com](http://www.widex.com)